

hatch and the case may be cleaned with a dry cotton bud once the Battery Cover (9) has been removed. Be sure that the rubber seal is free of dirt before replacing the Battery Cover, as this could compromise the seal.

- When not in use, keep your Hero in its protective case. Do not store in a damp area.

11. TROUBLESHOOTING

1. Check incorrect fitment and placing of Hero on baby's diaper (see Section 5 *Diaper/Nappy Fitment* and Section 6 *Fitting your Hero*).
2. Battery failure. The Battery Indicator Light (2) will show red (see Section 9.3 *Replacing the Battery*).

12. DISCLAIMER

Customers use this monitor at their own risk and Biosentronics cc, its members, employees, agents, contractors, representatives, manufacturers, distributors and associates ("Snuza") shall not be held liable for any loss, damage or injury suffered by the customer or any other person arising from, or in connection with, the use or manufacture of the monitor, including loss, damage or injury suffered by the customer or any other person as a result of any defects in respect of the monitor, and all warranties, and any other liability from whatsoever cause arising, are hereby excluded and the customer hereby indemnifies Biosentronics cc in respect of any such liability.

13. GUARANTEE

Biosentronics cc guarantees this product to be free from defects in materials and workmanship for a period of 12 months from the original date of purchase, verified by a sales receipt.

Our guarantee liability shall be limited to replacing the unit free of charge and does not include transport costs. Our guarantee extends to all countries in which this product is supplied. By the act of operating this product, the user accepts all resulting liability. We reserve the right to modify the provisions of this guarantee at any time without notice. This guarantee does not cover the battery, incorrect application, tampering, misuse or transport damage.